

Drive Nebulizer

Quick Reference Guide



Parts & Accessories



Compressor



Disposable Cups
& Tubing



Re-useable
Cup & Tubing



Filter



Facemask

Assembly & Use

1

Place the Compressor on a table or other flat, stable surface.

Do not use this device on the floor.

2

With the power switch in the O (OFF) position, plug the power cord into an appropriate electrical wall outlet.

3

Unscrew the cap of the nebulizer cup. Add the prescribed medication to the cup & reattach the cap.

4

Add the T-Connector connector to the cup, and the mouthpiece to the t-connector.

5

Connect the reservoir tube to the other end of the t-connector. Connect the tubing to the air inlet connector found at the bottom of the nebulizer cup.

6

Turn on the Compressor by pressing the power switch to the I (ON) position and begin treatment. The machine should make a humming sound, and the machine should produce a medicated mist from the mouthpiece.

7

When the treatment is complete, turn off the compressor by pressing the power switch to O (OFF) position and unplug the unit from the electrical outlet.



Treatment is complete when there is no longer a visible mist coming from the tubing, and the cup begins to make a sputtering sound.

Cleaning & Storage

1

With power switch in the O (OFF) position, unplug power cord from wall outlet.

2

Disconnect tubing from the air-inlet connector and set aside.

3

Disassemble mouthpiece (or mask) from cap. Open the nebulizer by turning cap counter-clockwise.

4

Wash all items, **except tubing**, in a hot water/soapy solution. Rinse under hot tap water for 30 seconds and allow to air dry.

Remember to wash after every use!

Store nebulizer in a cool, dry place out of reach of children and pets.